Etac Torkel toilet paper tongs

To manage intimate hygiene is a high priority task for everyone.

Improved pricacy

When grip, reach or mobility functions are restricted, these tongs can be used for toilet paper, wash cloths or small towels. Torkel can be disinfected and also withstands boiling.

How to use Torkel toilet paper tongs:



1. Tear off a piece of toilet paper (approx. 40 cm / 16"). Open the tongs by squeezing the handle, then thread the paper between the tongs.



2. Roll the paper loosely around the tongs so that it covers the tip.



3. Wipe.



4. Open the tongs and shake to remove the toilet paper.





DescriptionTorkel toilet paper tongs

Item no. 80304001

Dimensions

Length: 33 cm (13")

Material

Polyamide plastic

Weight

100 g (3.5 oz)

Design

Designkonsulterna







Clean the product with a solvent-free cleaning agent (pH level 5–9), or with a 70% disinfectant solution. Can be decontaminated at max. 85°C / 185°F if so required. Also withstand boiling.