



Care Quip

FOLDING INSTRUCTIONS

NC1050 Vito Plus Wheelchair



Care Quip Pty Ltd

5 Millennium Court Silverwater NSW 2128

P 02 9714 1111 E info@carequip.com.au

www.carequip.com.au

Welcome

Congratulations on choosing to buy a Care Quip quality product. All Care Quip products are manufactured to the highest standards of performance and safety.

As part of our philosophy of customer service and satisfaction, Care Quip products are backed by our comprehensive One Year Warranty. We hope you will enjoy using your Vito Plus Wheelchair.

To Fold the Wheelchair

- Pivot footplates upward to vertical position.
- With both hands, grasp the middle of the seat upholstery at the front and back edge and lift up.
- Place excess seat upholstery over the armrest.



Warning

Do not allow upholstery to hang between the cross braces.

To Unfold the Wheelchair

- Stand and face the side of the wheelchair
- Tilt the wheelchair toward you (raising the opposite wheel and caster off the ground/floor).
- Place your hand on top of the seat rail closest to you where the seat upholstery is attached.
- Point your fingers and thumb to the inside of the wheelchair.
- Press downward on the top of the seat rail until the wheelchair is fully open and the seat rails are fully locked into the U-shaped seat locations brackets (see pic below). Failure to do this could result in wheelchair frame misalignment
- Engage both wheel brakes
- Open the footrest/legrest for clearance and transfer into the wheelchair.

